


Imagescan HD

Adjusting images to be lighter or darker

Step 1: Is Extension Cone being used?

If Yes: Panel should show an "L" (for Long Cone) next to the cone icon (vs. "S" for short as shown). Setting to L will increase power across all modes and will make images darker. Change this setting in the configuration menu (see next page).

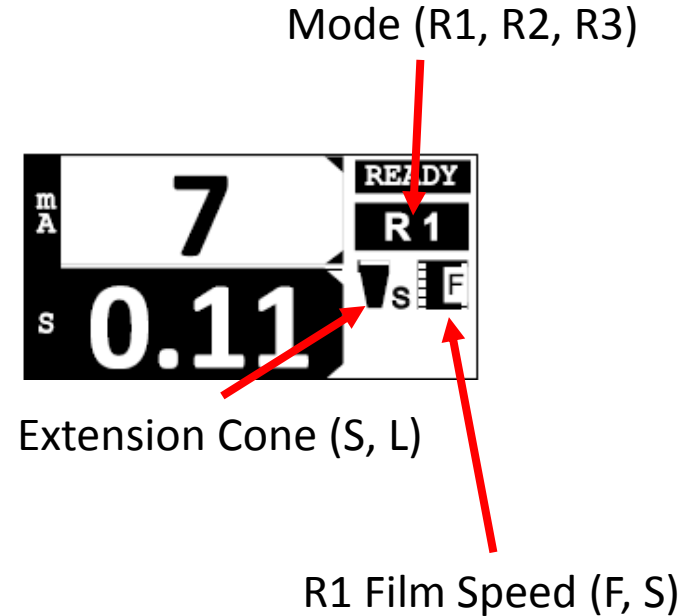


Step 2: Are images still too light or too dark?

Adjust the Mode in the configuration menu (see next page)

- Receptor 1 (R1)
 - Slow (S)
 - Fast (F)
 - Receptor 2 (R2)
 - Receptor 3 (R3)
- ↑ Raise power (R1, Slow) to make images darker)
- ↓ Lower power (R2, R3) to make images lighter)

To change settings listed above:

1. Turn off unit and turn on
2. When ImageWorks logo appears, simultaneously press the up and down key and release (if buttons are held down, the menu will not appear). This will access the configuration menu
3. Menu tree is shown below
 - Press up or down keys to navigate within each menu
 - Press  key to select an option
 - To return to previous menu, select <back option or press Adult/Child key

