



How to Bypass the Daughter board with the Imagescan HD

Bypassing the Daughterboard can sometimes be a useful tool during troubleshooting to isolate certain problems. For instance, if a particular error goes away when the daughterboard is bypassed, this may indicate that the problem lies either in the daughterboard itself or connections to/from the daughterboard.

1. Turn unit off
2. Underneath the main console assembly, disconnect the RJ45 connector that is on the left (when facing the unit).

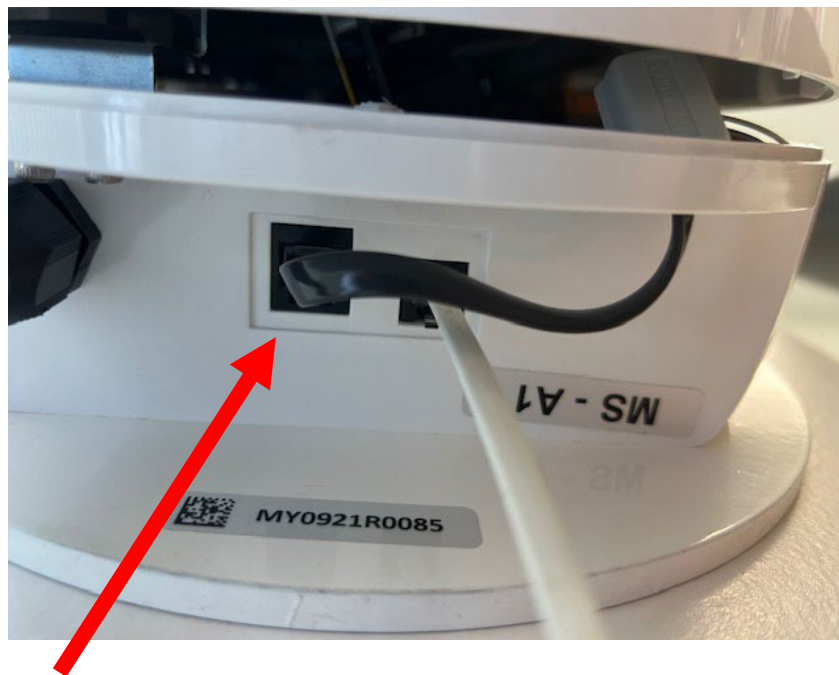

3. Open the front cover, and locate the cable that runs from the inside of the cover to the J5 connector.

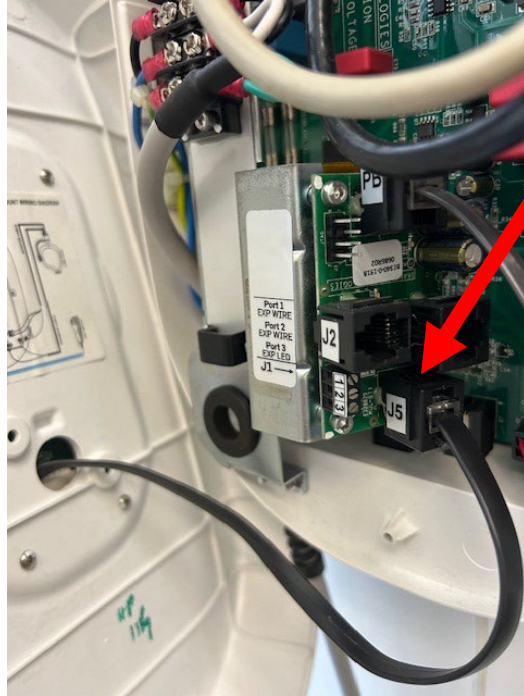

4. Disconnect the cable from the J5 connector.
5. Take this end of the cable and plug it into the RJ45 connector underneath the main console assembly (that was vacated in step 1).
6. Turn unit on

Do not forget to return the unit to its normal configuration when finished troubleshooting.