

How to Replace Elbow Covers of Imagescan HD or Intraskan DC

Remove Old Covers

To replace the plastic elbow covers, the rubber slot cover must first be pulled out. To do this, simply use your finger to extract this.



Remove old elbow covers



With the new joint covers, the original rubber slot cover is slightly too large, and needs to be trimmed to fit inside the new covers.

Please note: The rubber slot covers are purely aesthetic and are not critical to the operation of the unit. Therefore, there are two options: the slot covers can simply be removed or they can be trimmed to fit into the new covers.

Trimming Slot Covers (Optional)

Depending on which joint cover is being replaced, the rubber slot cover will look slightly different. For the joints at the midsection of the elbow arm, they look like this:



To trim this cover, cut about $\frac{1}{4}$ " off the longer flap, and trim the corners for better fit





For either the joint near the head or the joint near the straight arm, the slot cover will look like this:




To trim this cover, cut about 1/2" off the end with the two tabs, and trim the corners for better fit




Place new covers

Before placing the new covers, it's worth orienting yourself to see how the tabs connect.

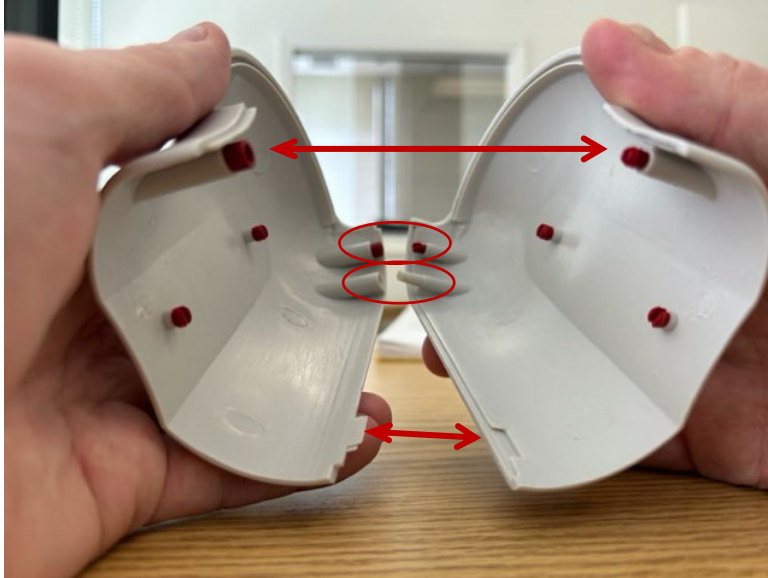

Then hold the covers like this with rounded edges facing down

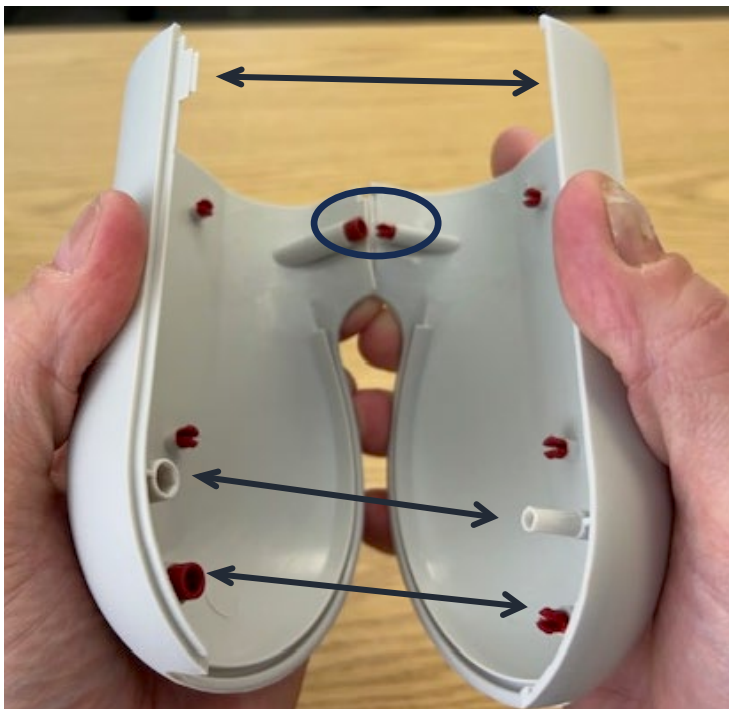

Place covers around the elbow and snap the connections together to hold in place



Tuck the slot covers into place (if they have not been removed)

