

Sensibles Sensor Positioning System

Cleaning and Sterilizing Options (autoclave before first use):

1. Soak for one hour in glutaraldehyde. DO NOT USE PHENOL-BASED GLUTARALDEHYDE. Rinse and dry.
2. Clean ultrasonically, use a general purpose ultrasonic cleaner or detergent/soap and water. DO NOT USE SPECIAL APPLICATION CLEANERS. Rinse and dry.
3. Steam Autoclave (distilled water only). DO NOT CHEMICLAVE OR DRY HEAT STERILIZE. DO NOT USE THE AUTOCLAVE DRYING CYCLE. OR Cold Sterilize. CHEMICALS MAY CAUSE SKIN IRRITATION IN SOME PATIENTS