

# **ImageScan HD X-ray**

## **How to adjust parameters**


- 1. How to make images lighter or darker (changing mode)**
- 2. How to save mA or time settings**
- 3. How to change kV settings**



# 1. Adjusting images to be lighter or darker

## Step 1: Is Extension Cone being used?

If Yes: Panel should show an “L” (for Long Cone) next to the cone icon (vs. “S” for short as shown). Setting to L will increase power across all modes and will make images darker. Change this setting in the configuration menu (see next page).

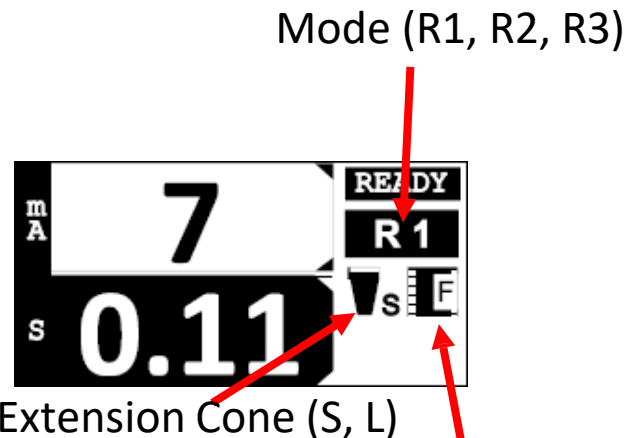

## Step 2: Are images still too light or too dark?


Adjust the Mode in the configuration menu (above how to access)

- Receptor 1 (R1)
  - Slow (S)
  - Fast (F)
- Receptor 2 (R2)
- Receptor 3 (R3)



Raise power (R1, Slow) to make images darker)

Lower power (R2, R3) to make images lighter)

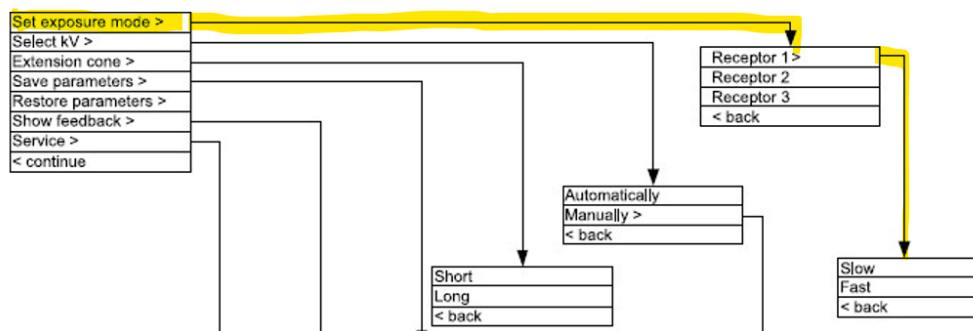


## 2. How to save mA or time settings

Access the configuration menu  
(see above)

Select Save parameters and  
choose “Always”.

This will cause changes to mA or  
time to remain

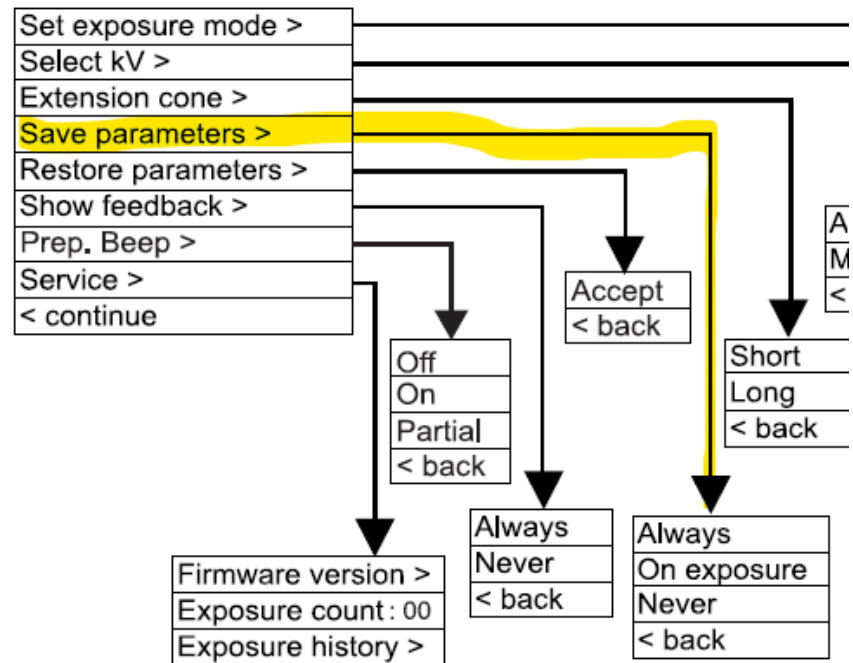

### 3. How to change kV settings

Access the configuration menu  
(see above)

Select kV and select Manually

Adjust kV to desired value

