

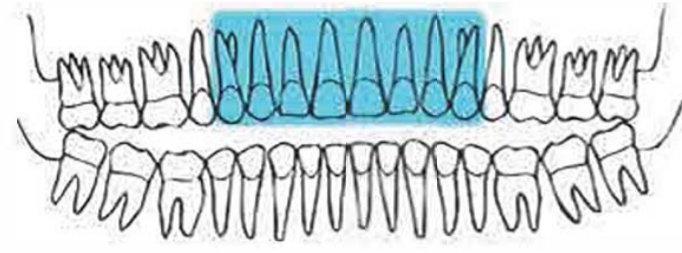


## Sensibles Anterior Positioning Guide

### Maxillary

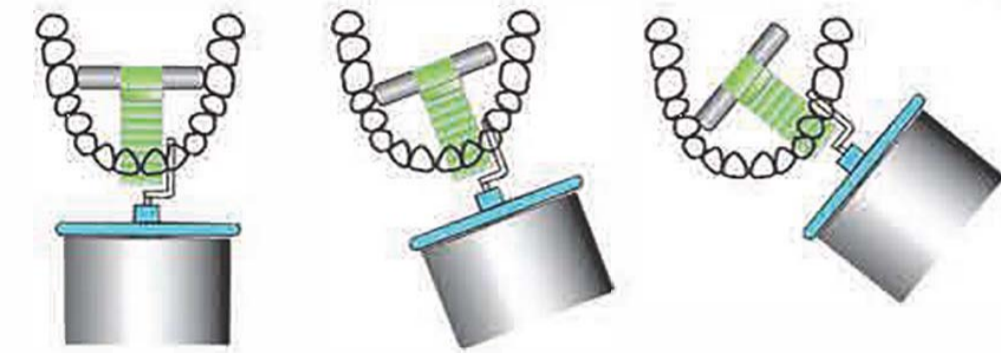


Use either Bite Block, depending on size of sensor.


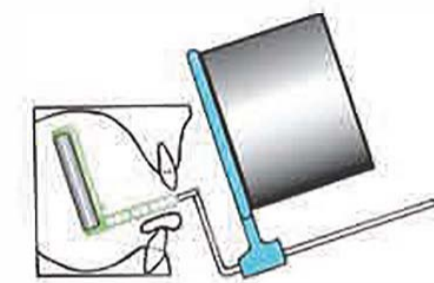


Use Alignment Arm "ANTE 40857-10."

### Mandibular

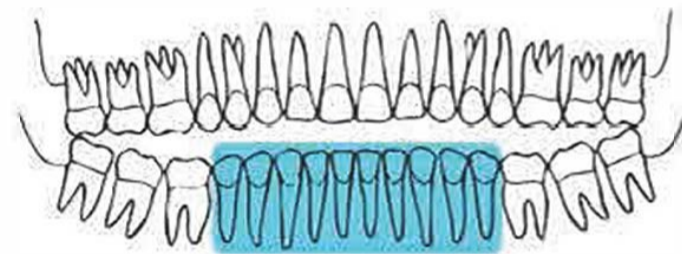



Use either Bite Block, depending on size of sensor.

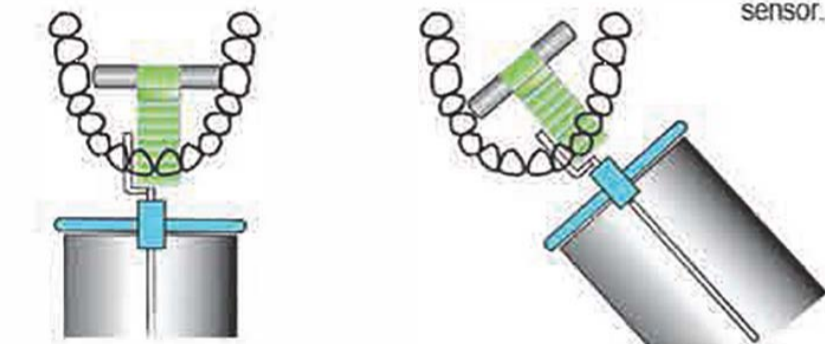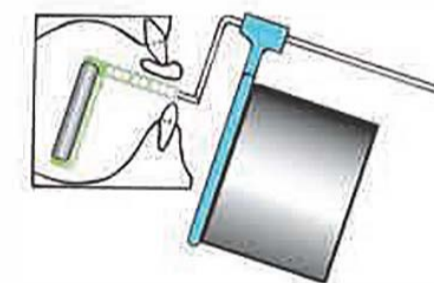

Use Alignment Arm "ANTE 40857-10."

## Sensibles Posterior Positioning Guide



### Maxillary

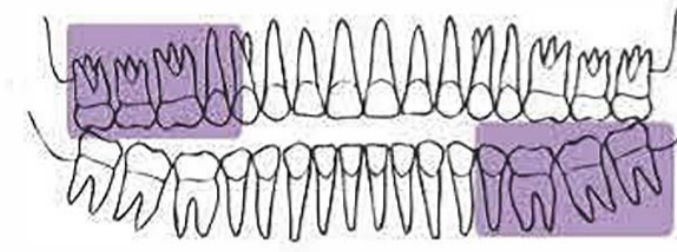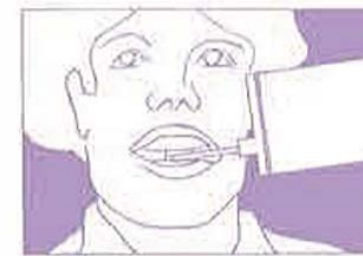



Use either Bite Block, depending on size of sensor.



Use Alignment Arm "POST 40858-10."

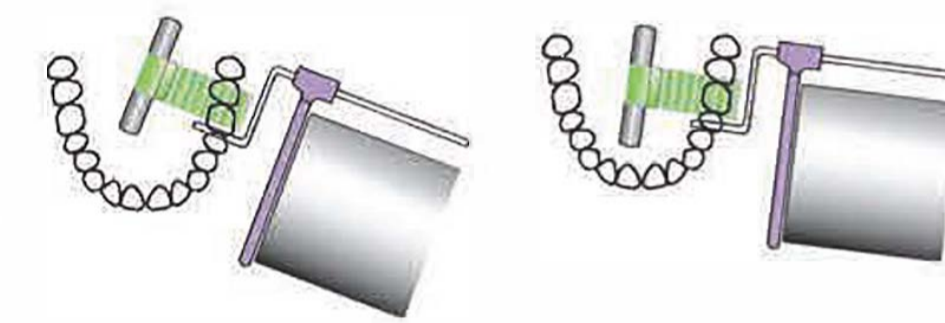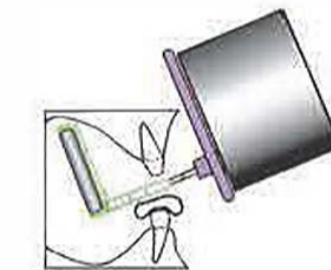



### Mandibular

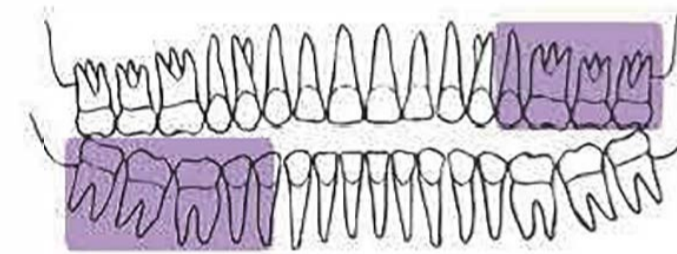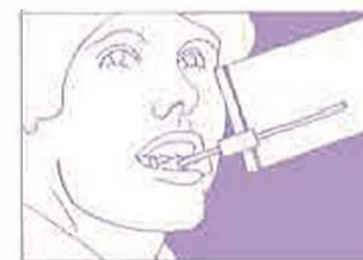



Use either Bite Block, depending on size of sensor.



Use Alignment Arm "POST 40858-10."

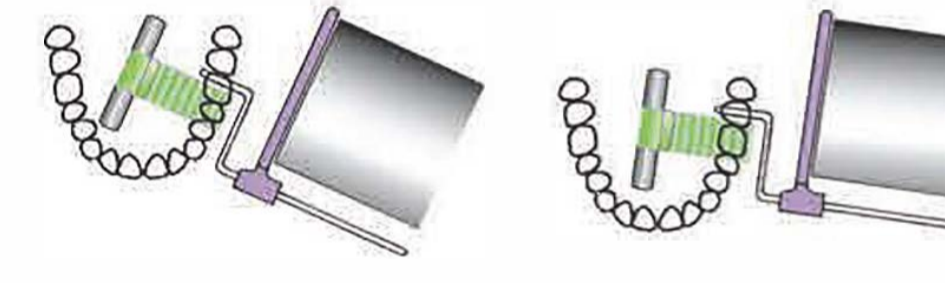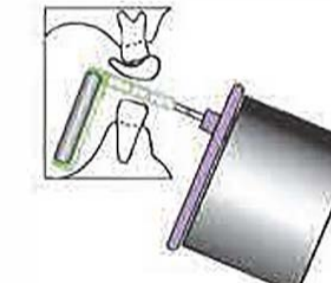



## Sensibles Bite-Wing Positioning Guide

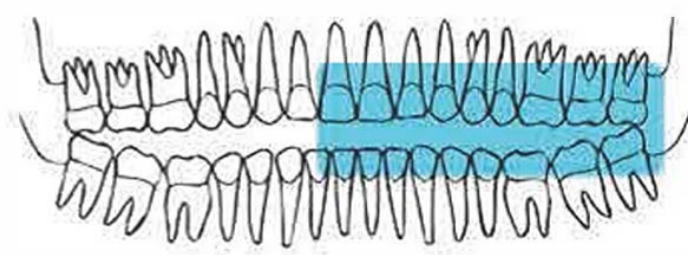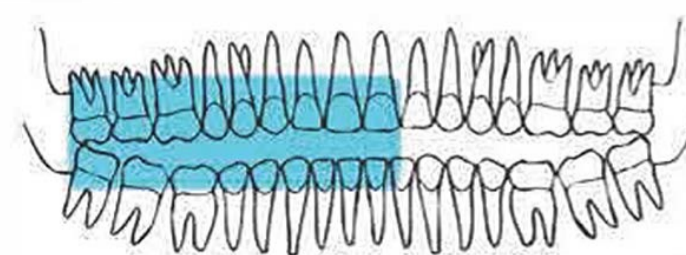

Use either Bite Block, depending on size of sensor, then simply slide Bite Block to mid-line of sensor.



Use Alignment Arm "BITE 40915-10."

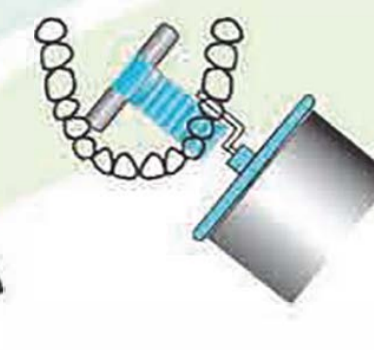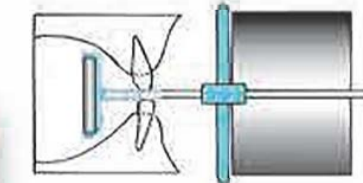

### Horizontal Imaging



The Medium Sensible holds most size 2, 1 and 0 sensors horizontally.

### Vertical Imaging



The Large (two-tone) Sensible holds most size 2 and 1 sensors vertically.

### Bite-Wing Imaging



For Bite Wing imaging, simply slide Bite Block to mid-line of sensor.

### To Insert Sensor



1. Place sensor in holder



2. Push top down

### To Remove Sensor



Push button on back

### We Recommend



Large: 80210 Medium: 80200